

DMC-EZ

EzDicom Converter

EzDicom Converter lets you scan documents, films, import JPEGs, and capture analog video frames, convert to DICOM and send to any PACS. EzDicom Converter simple user interface allows anyone to scan films and import JPEGs, or capture video frames.

Why EzDicom Converter?

- Easily scan documents or films and send to PACS.
- Import JPEGs, BMP's and Video Frame in less then one minute and convert to DICOM.
- Automatic demographic match and reconcile with PACS or worklist.

The screenshot shows the 'Study Demographic' dialog box with the following fields and values:

- Patient Demographics:**
 - Patient ID: 20100507152118
 - Name: [Empty]
 - Sex: [Dropdown]
 - Date of birth: [Empty]
 - Age: [Dropdown]
- Study Demographics:**
 - Date Time: 05/07/2010 03:21 PM [Now]
 - Accession Number: [Empty]
 - Study ID: [Empty]
 - Modality: [Dropdown]
 - Description: [Empty]
 - Referring Physician: [Browse]
 - Body Part: [Browse]


Step 1: Select patient/study demographics from your RIS or PACS or manually enter the information



Sales Support: 800.944.6281
Outside US 408.350.3450
fax: 408.350.3100

309 Laurelwood Rd. #20
Santa Clara, CA 95054
www.sstgroup-inc.com
sales@sstgroup-inc.com

Step 2: Scan the film or document and click "Send to Destination"

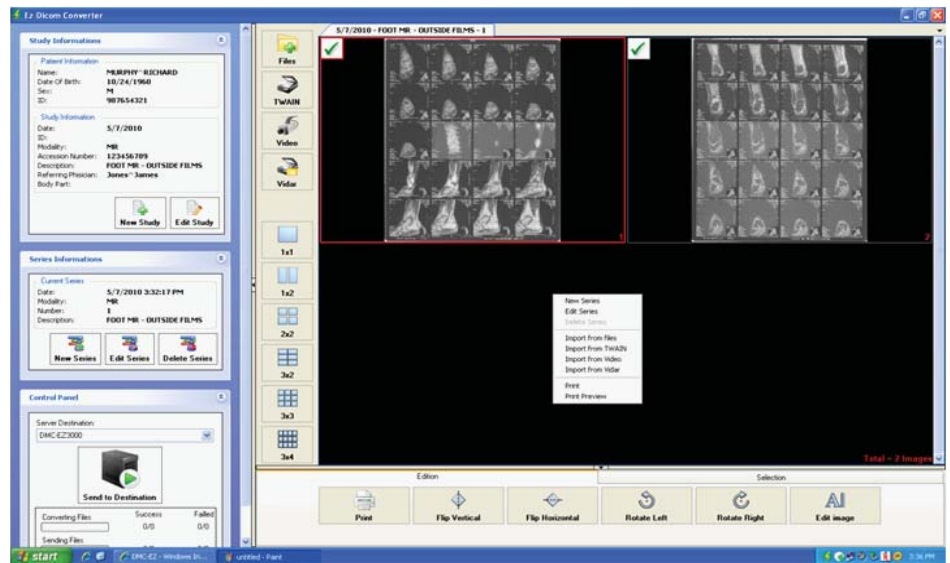

Done: The film or document is now matched with the appropriate study

Our intuitive Software is easy to use and requires little training or handholding. EzDicom Converter software will automatically populate patient demographic information with interfaces to the RIS worklist and PACS. Scan documents and films and send to PACS for inclusion with the patient record.

Features at a Glance

- Multi-purpose DICOM Converter
 - Import non DICOM Images (JPG, BMP)
 - Import images from Twain compliant scanners (Reports, Notes, etc)
 - Interface with Vidar Film Digitizer to receive scanned films
- Search feature allows you to find matching Patients in your Worklist(s) and/or PACS and do the reconciliation process
- Select desired images and send to DICOM destination
- Configuration of an unlimited number of DICOM destinations
- Automatic sending of images to a standard destination



Sales Support: 800.944.6281
Outside US 408.350.3450
fax: 408.350.3100

309 Laurelwood Rd. #20
Santa Clara, CA 95054
www.sstgroup-inc.com
sales@sstgroup-inc.com